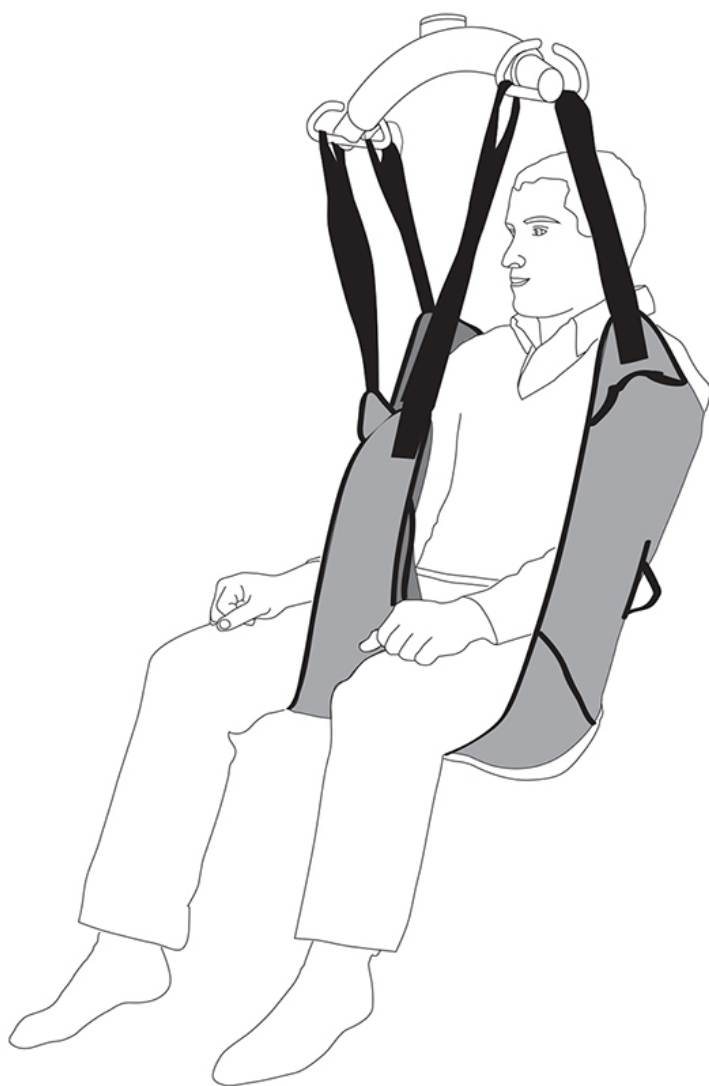
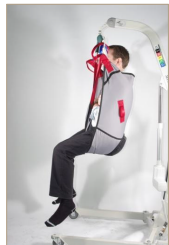

Duracare[®]
Patient Lifting Slings



- Every batch tested for quality assurance
- Woven label – no fading
- Hard wearing fabric or mesh available
- All approved to 300kg capacity
- Smart Silver Material makes coloured straps stand out

- Premium Quality Made in New Zealand
- Exceed test requirements
- Quick and easy ID coloured straps e.g. Red = Medium
- Can be used with any appropriate lifter
- Washed up to 95° - very hygienic



General Purpose



General Purpose with Head Support



Head Support Hygiene

Duracare Sling

CUSTOM SIZES, CUSTOM SLING

Made to order, tell us what you need..

ALSO STOCKED:

- Pivot Slings
- Pivot Hygiene Slings
- Hammock Slings
- Hygiene Buckle Waist Band
- Hygiene Velcro Waist Band
- Paediatric Slings
- Stand-Up – Velcro and buckle, loop & keyhole Slings
- Unique Pringle Stand-Up Sling

COLOUR CODING:

Strap colour indicates the size..

XS	S	M
L	XL	XXL

SLINGS - SIZING CHART

Sizing & Body Measurement

STEP 1: Torso Height

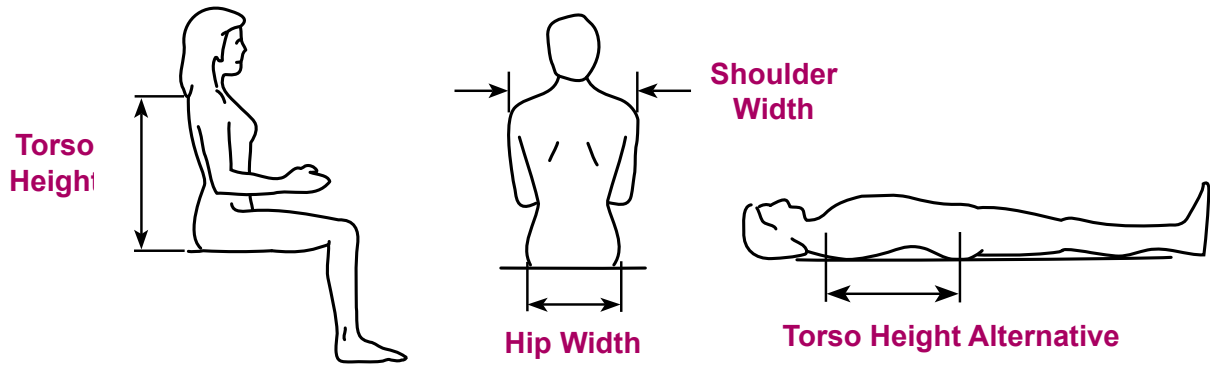
This relates to the 'shoulder sitting height' which is measured as the vertical distance from a hard horizontal surface to the acromion (the bony point on top of the shoulder) while sitting erect. As some clients may be unable to be measured in this posture an equivalent measurement can be taken from the greater trochanter (the bony prominence at the side of the hip) to the acromion while lying horizontal.

STEP 2: Hip Width

This is measured as the distance across the hips in a sitting position.

STEP 3: Shoulder Width

This is not usually as critical as hip width when correctly fitting a sling. However, if the client's shoulder width is relatively small when compared to hip width, it will be hard to achieve a good upright position when lifting for transferring to a chair or toilet. It is measured across the shoulders at the outer points of the deltoid muscles.



SIZE	2XS	XS	S	M	L	XL	2XL	3XL
cm	29 - 32.5	32.5 - 36.5	36.5 - 41	41 - 46	46 - 51	51 - 56	56 - 61	61 - 67
inches	11.4 - 12.8	12.8 - 14.4	14.4 - 16.1	16.1 - 18.1	18.1 - 20.1	20.1 - 22	22 - 24	24 - 26.4

SLINGS - SIZING CHART

PLEASE NOTE: This is a guide only. Whilst this calculator is aimed to simplify sling sizing selection, the sling must always be checked before use and the appropriate style used.

TORSO HEIGHT

	30 cm 12 in	34 cm 13.5 in	38 cm 15 in	42 cm 16.5 in	46 cm 18 in	50 cm 20 in	54 cm 21.5 in	58 cm 23 in	62 cm 24.5 in	66 cm 26 in	70 cm 27 in	74 cm 29 in	78 cm 30.5 in	82 cm 32 in
21 cm 8.3 in	2XS	2XS	2XS	2XS TALL	2XS TALL									
23 cm 9.1 in	2XS	2XS	2XS	2XS TALL	2XS TALL									
25 cm 9.8 in	2XS	2XS	2XS XS	XS	XS	XS TALL	XS TALL							
27 cm 10.5 in	XS SHORT	XS SHORT	XS	XS	XS	XS TALL	XS TALL							
29 cm 11.4 in	XS SHORT	XS SHORT	XS	XS	XS S	S	S	S TALL	S TALL					
31 cm 12.2 in			S SHORT	S SHORT	S	S	S	S TALL	S TALL					
33 cm 13 in			S SHORT	S SHORT	S	S	S M	M	M	M TALL	M TALL			
35 cm 13.8 in					M SHORT	M SHORT	M	M	M	M TALL	M TALL			
37 cm 14.6 in					M SHORT	M SHORT	M	M	M L	L	L	L TALL	L TALL	
39 cm 15.4 in							L SHORT	L SHORT	L	L	L	L TALL	L TALL	
41 cm 16.1 in							L SHORT	L SHORT	L	L	L XL	XL	XL	XL TALL
43 cm 16.9 in									XL SHORT	XL SHORT	XL	XL	XL	XL TALL
45 cm 17.7 in									XL SHORT	XL SHORT	XL	XL	XL 2XL	2XL
47 cm 18.5 in										2XL SHORT	2XL SHORT	2XL	2XL	2XL
49 cm 19.3 in										2XL SHORT	2XL SHORT	2XL	2XL 3XL	3XL
51 cm 20.1 in										2XL SHORT	2XL SHORT	3XL	3XL	3XL
53 cm 20.9 in										2XL SHORT	2XL SHORT	3XL	3XL	3XL

IMPORTANT: This special sizing chart only applies to Duracare Sling Range.

Through our innovative, quality products, exceptional service and ingrained knowledge and expertise of the healthcare industry, Highgate Healthcare has been raising the standard of living in aged care, hospital, disability and rehabilitation facilities in Australia for over 30 years.



Highgate Healthcare

1300 350 350

2 Selgar Avenue Tonsley SA 5042

highgatehealthcare.com.au