

INSTALLATION: CONFIGURA COMFORT HANDSET HOLDER

Contents

- 1x Handset Holder
- 3x Screws

Required Tools


- Drill
- Phillips #2 screw bit

Installation

Prior to installing ensure you have selected the correct version (left or right-handed) of the Handset Holder according to side of the chair being used.


Mount the handset holder

1.

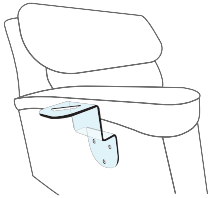

Position the handset holder at a comfortable distance from the backrest. Ensure that the top surface of the holder is flush with the top of the armrest.

2.

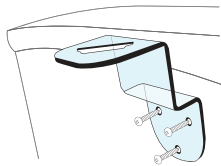


Fasten the handset holder to the side of the chair using the screws. Start by fastening the bottom screw then fine-tune the position of the handset holder before fitting the remaining screws.

All screws should be firmly tightened.

Install the handset

3.

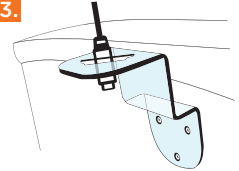

Unplug the handset and insert the cable through the opening in the handset holder from above.

Re-plug the handset.

Cautions and Warnings

- Only use the screws provided
- Do not lean on the Handset Holder as it is not designed to be weight bearing
- Must be installed by a competent person

Cleaning

- ✓ Only clean the handset holder with warm water and a soft cloth
- DO NOT USE abrasive cleaners or scourers as these can scratch the surfaces
- DO NOT USE solvents and cleaning products (*such as Windex*) as these can cause the Handset Holder to discolour